

hscTnI Rapid Quantitative Test



INTENDED USE

The Biotime hscTnI (highly sensitive cardiac troponin I) Rapid Quantitative Test is intended to quantify the concentration of cTnI in human serum, plasma or whole blood on Biotime FIA Analyzers (Semi-automatic / Automatic) by fluorescent immunoassay. It is used as an aid detection of myocardial Infarction.

-Fluorescent immunoassay

-Myocardial Infarction

For in vitro diagnostic use only. For professional use only.

INTRODUCTION

This product is used to detect concentration of cardiac troponin I (cTnI) in human serum, plasma or whole blood. The troponin complex, consisting of cTnI, troponin T (TnT) and troponin C (TnC), can adjust the interaction between actin and myosin. Concentration of cTnI is pretty low in healthy human blood. However, when acute myocardium infarction occurs, cTnI is released into blood quickly, which leads to rapid increase of cTnI concentration in hours and last 6 to 10 days. Therefore, blood cTnI concentration is regarded as a marker for myocardial infarction^[1-3].

PRINCIPLE

This test kit is based on fluorescent lateral flow immunoassay. While the sample and the buffer are mixed and applied into the test cartridge, the cTnI in the sample and the mouse anti-cTnI monoclonal antibody labeled with fluorescent microsphere form a reaction intermediate complex. During lateral flow, the intermediate complex moves along the nitrocellulose membrane to a detection line (T-line: coated with cTnI specific monoclonal antibodies). The intermediate complex will be captured by T-line to form final reaction compound sandwich. Thus the fluorescent signal on detection line is positively correlated with the concentration of cTnI in human blood.

The fluorescent signal from microspheres of compound sandwich will be detected and calculated according to the calibration curve (in SD card provided with the reagents) to represent the concentration of cTnI in human blood.

PRECAUTIONS

1. This reagent is used for in vitro diagnosis only. Please do not use expired products.
2. All samples (including the remaining samples after testing), used reagents and waste should be treated as infectious materials.
3. The reagent is for single use.
4. Once opening, the reagent should be used within the specified time to avoid detection failure due to moisture absorption.
5. While using the test cartridge and instruments, vibration and strong electromagnetic environment should be avoided.
6. Lot number of buffers and test cartridges must be matched.
7. Do not insert the cartridges that are contaminated with blood or other liquids on the surface. Otherwise, it may cause damages to the instrument.

MATERIAL

Material Provided

The following components are included in the hscTnI Rapid Quantitative Test.

Specification and Component

1. For type A reagent

Packing specification				
Catalog No.	Test cartridge	Detection buffer	SD card	Instructions for Use
BT0201201	25 tests/kit	25 tubes/kit	1 piece/kit	1 copy/kit

2. For type B reagent

Packing specification				
Catalog No.	Test cartridge	Detection buffer	SD card	Instructions for Use
BT0201202	25 tests/kit	1 panel/25 tests	1 piece/kit	1 copy/kit

Material Required But Not Provided

1. Biotime FIA Analyzer

Applied information	Biotime FIA Analyzer	Applicable model
For type A reagent	Semi-automatic	BIOT-YG-I, FLI-100, FLI-600, FLI-1200
For type B reagent	Automatic	FLI-4000

2. Transfer Pipette Set (range 5-50μL and 10-100μL size)
3. Specimen Collection Containers
4. Timer

STORAGE AND STABILITY

1. Store the detection buffer at 2-30°C, and the shelf life is 24 months.
2. Store the test cartridge at 2-30°C, and the shelf life is 24 months.
3. For Type A reagent: Test cartridge should be used within 30 min after opening the pouch. For Type B reagent: After opening the kit, it should be exposed to the air for no more than 24 hours under the conditions of 10-30°C and 30%-90% relative humidity. It is recommended to use the original self-sealing pouch to seal the unused test cartridge and use it within 7 days. And the sample buffer should be ready for use.

SPECIMEN COLLECTION AND PREPARATION

1. The test can be performed with human serum, plasma or whole blood specimen only.
2. The specimen collection container should be procoagulant tube, heparin anticoagulant tube or EDTA tube.
3. The collection of the sample: the venipuncture for blood collection method referring to the National Clinical Laboratory Procedures, if the sample can't be detected in time, it can be stored in refrigerator at 2-8°C for no more than 7 days, or at -20°C for no more than 6 months. Samples must be recovered to the room temperature before test.
4. Separate plasma as soon as possible to avoid hemolysis.

TEST PROCEDURES

Please refer to the operation manual of Biotime FIA analyzers for details. The test should be operated at room temperature (~25°C).

For Type A reagent

Step 1: Preparation

Check the lot number and insert SD card into the equipment. Take out one tube of buffer from refrigerator and balance it to room temperature.

Step 2: Sampling

Take 40μL of serum/plasma or 60μL of whole blood with a transfer pipette and add it into the buffer tube.

Step 3: Mixing

Mix well the specimen with buffer by tapping or inverting the tube.

Step 4: Loading

Take 80μL of sample mixture and load it into the well of the test cartridge.

Note: Step 2 to step 4 should be completed within 1 minute to ensure the accuracy of the test results.

Step 5: Testing

Ensure that there are no air bubbles. Immediately insert the test cartridge into analyzer and incubate for 15 minutes.

For Type B reagent

Step 1: Preparation

Check the lot number. Tear the aluminum foil bag, take out the test cartridge and buffer, and put it into the analyzer card slot.

Step 2: Sampling

For Serum or plasma samples, after put the samples to be tested into the analyzer, the analyzer automatically absorbs 40μL samples.

For whole blood sample, after put the sample to be tested into the analyzer, the analyzer automatically absorbs 60μL sample;

Step 3: Mixing

The instrument automatically adds the sample into the sample buffer, and then fully mixed.

Step 4: Loading

The instrument automatically absorbs 80μL of the above mixed solution into the detection hole of the test cartridge, and moves it to the incubator;

Step 5: Testing

After sample mixture addition, the test cartridge reacts in the incubation tank for 15min, and the analyzer reads the result;

NOTE: Please refer to the operation manual of a specific model of the analyzer for details.

REFERENCE INTERVAL

Normal reference interval: <40pg/mL

Note: Individual reference range is suggested to be established for each laboratory.

LIMITATIONS OF PROCEDURE

1. The test sample should be serum, plasma or whole blood.
2. Human anti-mouse antibody (HAMA) may be presented in patients who have received immunotherapy with a murine monoclonal antibody. This kit has been specially designed to minimize the effect of these antibodies on the test results. However, the test result must be

carefully evaluated when patients are known to have these antibodies^[4].

- Other factors also can induce the false results, including the technology, operational error and other sample factors.



Xiamen Biotime Biotechnology Co., Ltd.
Address: 2F/3F/4F/5F, No.188, Pingcheng South Road, Haicang Street, Haicang District, Xiamen City, Fujian Province, 361026, P.R. China

PERFORMANCE CHARACTERISTICS

Accuracy

cTnI control materials with two different concentrations were tested by every lot of Test Cartridges, and the deviations were within $\pm 15\%$.

Assay Range: 20-40000pg/mL

Limit of blank: ≤ 10 pg/mL

Limit of detection: 20pg/mL

Limit of quantitation: 20pg/mL

Linearity

A serial concentrations of cTnI reference materials at 20-40000pg/mL were tested, and the correlation coefficient (R) is ≥ 0.9900 .

Precision

Intra-lot Precision

Intra-lot precision C.V. is $\leq 15.0\%$

Inter-lot Precision

Inter-lot precision was determined by testing of cTnI reference materials. The C.V. is $\leq 15.0\%$.

Specificity

The following substances were not found to cross-react with cTnI in this concentration range.

Number	Antigen	Concentration of antigen	Test result of hscTnI
1	sTnI	1000 μ g/L	<40pg/mL
2	cTnC	1000 μ g/L	<40pg/mL
3	cTnT	1000 μ g/L	<40pg/mL

SYMBOLS

Symbol	Description	Symbol	Description
	Catalogue number		In vitro diagnostic medical device
	Batch code		Consult instructions for use
	Date of manufacture		Keep dry
	Use-by date		Keep away from sunlight
	Manufacturer		Temperature limit
	Do not re-use		Authorized representative in the European Community/European Union
	CE mark		

BIBLIOGRAPHY OF SUGGESTED READING

- Le Tao. The clinical application and evaluation of the Triad rapid diagnosis to myocardial infarction. Journal of Chinese Clinical Medicine. 2005, 140: 15239- 15240.
- Albert JS, Thygesen K, Antman E, Bassand JP. Myocardial infarction redefined—a consensus document of the Joint European Society of Cardiology/American College of Cardiology Committee for the Redefinition of Myocardial Infarction.
- Bernadette Cummins B. Sc. Margaret Lucy Auckland B. Sc. Peter Cummins Ph.D. Cardiac-specific troponin-I radioimmunoassay in the diagnosis of acute myocardial infarction[J]. Journal of American Heart. 1987, 113:1333-1344.
- Hansen JH. HAMA Interference with Murine Monoclonal Antibody-Based Immunoassays [J]. J of Clinical Immunoassay. 1993.16: 294-299.



Lotus NL B.V.
Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague, Netherlands
Tel: +31644168999

Version: A/03
Issuing date: 2024-08-15